One Year Limited Warranty

Your Vicks VapoSteam Inhaler is warranted against defective materials or workmanship for one year from date of purchase. Any defective part will be replaced at no charge if it has not been tampered with and appliance has been used according to our printed directions. This warranty gives you specific legal rights, as well as other rights which may vary from state to state.

If you experience unsatisfactory operation, please call the Kaz Consumer Service Department at (800)427-VAPOR 1-2, or write to Kaz, Inc. Consumer Service Dept. 250 Turnpike Road, Southborough, MA 01772, explaining just how the inhaler is operating. Our Consumer Service Representative will advise you how to correct the problem yourself, or will ask you to return the Inhaler for repair or replacement. www.kaz.com.

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kaz

Manufacturing Quality Healthcare Products for Over 75 Years

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Kaz, Incorporated • One Vapor Trail, Hudson, NY, USA 12534 • www.kaz.com

The Vicks VapoSteam Inhaler is distributed by Kaz, Inc. under license from The Procter & Gamble Company, Cincinnati, OH 45202. Vicks and VapoSteam are registered trademarks of The Procter & Gamble Company.

Kaz, Inc. distribuye el Inhalador Vicks VapoSteam bajo licencia de The Procter & Gamble Company, Cincinnati, OH 45202. Vicks y VapoSteam son marcas registradas de The Procter & Gamble Company.

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IMPORTANT! READ AND SAVE THESE INSTRUCTIONS

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Congratulations on your purchase of the **Vicks VapoSteam inhaler**. Your new Inhaler provides all the benefits of vapor therapy therapy at home, at the office or when traveling.

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Directions for Use

Rinse unit out thoroughly before first use.

1. Remove top of unit from base by twisting in opposite directions and lifting off top.
2. Fill base to water line with hot (not boiling) tap water. NOTE: If you overfill, the unit will leak.
3. Open Vicks scent pad packet and add to the water. As an alternative, one teaspoonful of Vicks VapoSteam or Kaz Inhalant can also be used. Replace the top.
4. Breathe in the soothing Vicks vapors.

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Care and Cleaning

After each use, thoroughly wash and dry your inhaler. The unit is top shelf dishwasher safe.

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Use with Vicks Scent Pads or Vicks VapoSteam

Your inhaler was designed to work with Vicks Scent pads, Vicks VapoSteam or Kaz Inhalant. These will provide soothing menthol and eucalyptus vapors for soothing comfort for coughs and colds.

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©2004, Kaz, Incorporated • Made and printed in China / Fabricado e impreso en China / Fabriqué et imprimé en Chine