SoftHeat®

Deep Muscle Therapy
Terapia Muscular Profunda
Thérapie musculaire profonde

SoftHeat® Deep Muscle Therapy

- A. Washable Protective Cover
- B. Pad

Terapia Muscular Profunda SoftHeat^{MR}

- A. Cubierta Protectora Lavable
- B. Almohadilla

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IMPORTANT SAFETY INSTRUCTIONS

DO NOT DESTROY

⚠ WARNING: Follow instructions carefully. Misuse can cause severe burns.

1. **Do not** heat this product with any means other than a microwave oven.
2. **This product could be permanently damaged if heated for more than double the guideline times.** This could affect the warranty.
3. **If overheated,** your Pad will be too hot to touch. Allow it to cool before removing from the microwave. Check if the Pad has been damaged. If there is any sign of damage, do not use the product. It should be discarded.
4. **Do not** use while sleeping. **Do not** allow children to use or heat this product unless under supervision of an adult. **Do not** use on an unconscious person.
5. **Do not** use this product with liniment, salve or ointment preparations that contain heat-producing ingredients. Skin burns could result.
6. **This product should not be used by** diabetics, or by any person who may be insensitive to the application of direct heat to the skin's surface, or persons with any skin disorder. If in doubt, consult your doctor before use.
7. This product will produce moisture. **Do not** use it resting against your furniture etc., as the pad will make the surface wet.
8. **CAUTION: Do not** exceed 30 minutes per application. Recommended treatment time is 10-30 minutes.
9. Check the treatment area every 10 minutes to help prevent burns or frostbite.
10. Always consult your physician before beginning any hot or cold therapy.
11. If pain doesn't improve or becomes more severe or if new symptoms occur, consult your physician immediately.

IMPORTANT

**Please do not return this product to the store.
Contact Kaz Consumer Relations – we can help you.**

Call: 1-800-477-0457 weekdays 7:30am – 7:00pm EST

Email: consumerrelations@kaz.com

Write: Kaz, Incorporated, Consumer Relations Dept.
250 Turnpike Rd., Southborough, MA 01772 USA

Canadian residents write to us at:
Kaz Canada, Inc., Consumer Relations Dept.
510 Bronte Street South, Milton, ON L9T 2X6 CANADA

OPERATING INSTRUCTIONS

⚠ CAUTION: Follow these instructions carefully. Misuse can cause severe burns. Overheating the SoftHeat Pad in a microwave can damage or destroy product.

Before heating, the SoftHeat Pad should be at room temperature. If you store the SoftHeat Pad in the freezer, follow instructions for HEATING FROM THE FREEZER below.

Heating Instructions

1. Make sure the microwave oven is completely clean and free of any food debris or fluids.
2. Place the SoftHeat Pad in your microwave oven without the cover.
3. In a 1,000+ Watt microwave, heat for 1 1/2 minutes.
4. When heated, place the SoftHeat Pad in the cover and place it over the area of treatment.

Heating From Freezer

Place Moist Heat Pad in microwave at DEFROST setting or 30% power level for 3-5 minutes. Allow to stand 5 minutes, then follow normal heating instructions.

Reheating From Warm

If reheating while still warm, heat for only 1 minute and if more heat is needed, heat only for a further 30 seconds. The time for reheating from warm should not be more than 1 1/2 minutes in total.

Cold Usage

1. Place your SoftHeat Pad in a refrigerator or freezer inside a sealed plastic bag.
2. Allow 2 to 3 hours in the refrigerator or freezer before use.
3. Wrap SoftHeat Pad in a towel or use the cover supplied and place on the area of treatment.

COMMONLY ASKED QUESTIONS

- Q: Why use moist heat therapy?**
- A:** Moist heat therapy increases blood flow to the areas being treated, bringing additional oxygen and helping provide temporary relief from pain. The application of moist heat is especially useful in treating muscle spasms from strain and tension. It also helps reduce pain in non-acute injuries such as minor muscular stiffness, arthritis pain, some headaches, and back or neck pain. Moist heat is believed to transfer the heat deeper and quicker into tissues than ordinary dry heat.

This page left blank on purpose

Q: What are therapeutic microbeads?

A: Therapeutic microbeads is the name for beads that are designed to produce moist heat. The beads absorb moisture from the air and release it as clean, deep-penetrating, moist heat when heated in the microwave oven.

Microbeads are odor-free, dust free, and will stay mold and mildew free. Unlike grain and pellet products, Microbeads will not dry out, dissolve, or develop odors.

Without heating water or a drippy mess, Microbeads deliver moist heat therapy safely and effectively.

Q: My SoftHeat Pad has been slightly soiled in the microwave, how do I wash it?

A: The surface of the pad may be cleaned using a mild cleanser and a damp cloth. If the SoftHeat Pad absorbs an excessive amount of water from washing or immersing in water, it will not function and produce moist heat.

Q: Can I place the SoftHeat Pad in the microwave with the cover?

A: No. Do not put the cover in the microwave with your SoftHeat Pad.

Q: How long should the therapy last?

A: The usual recommended hot and cold therapy is 10 to 30 minutes at a time with 2 to 3 hours in between each therapy. The area should be checked every 10 minutes to help prevent severe burns or frostbite.

WARRANTY

This product is warranted against defective material or workmanship from the date of purchase. The length of the warranty is 2 years.

Any defective part will be repaired or replaced at no charge as long as it has not been repaired or tampered with and was used according to these printed instructions. Any charges incurred to ship the product back for replacement are the sole responsibility of the consumer. This warranty gives you specific legal rights as well as other rights, which vary from state to state.

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Escriba: Kaz, Incorporated, Consumer Relations Dept.
250 Turnpike Rd., Southborough, MA 01772 USA

Residentes canadienses escriban a:
Kaz Canada, Inc., Consumer Relations Dept.
510 Bronte Street South, Milton, ON L9T 2X6 CANADA

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© 2008