# SmartTemp® / SoftHeat®

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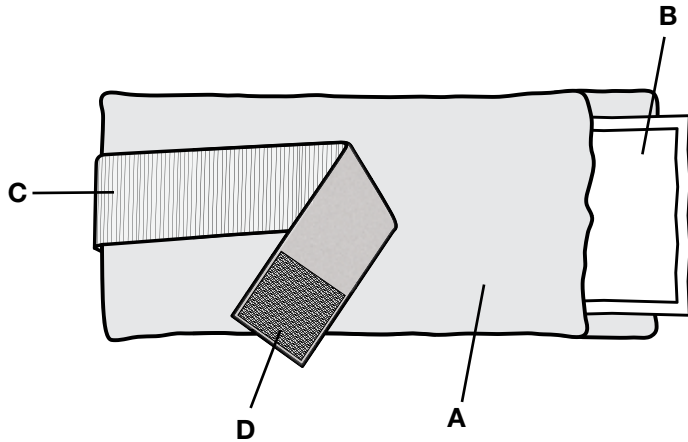
**CAUTION:** SmartTemp Portable Reusable Hot/Cold Pads and SoftHeat Back & Hip Wraps contain natural rubber latex which may cause allergic reactions.

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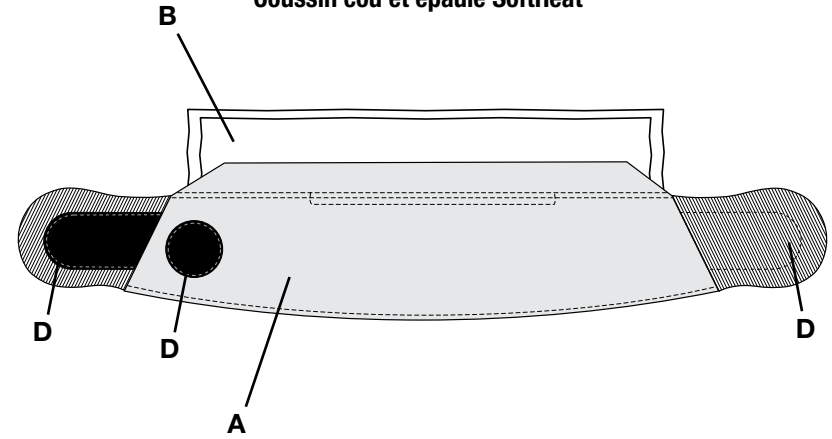
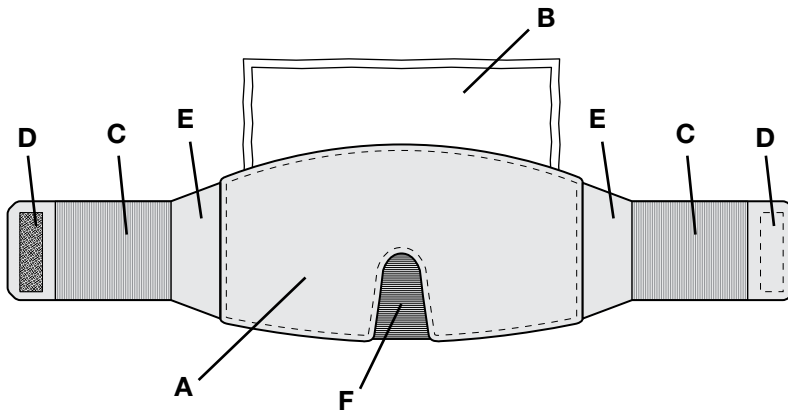
**NOTE:** Fastening Tips are designed only to fasten on the Elastic Straps or the Cloth Cover.  
 Wrap around back with "V" pointing down

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## IMPORTANT SAFETY INSTRUCTIONS

### DO NOT DESTROY

**⚠ WARNING:** Follow instructions carefully. Misuse can cause severe burns.

1. Do not apply the inner pack directly to the skin. Always use the cloth cover.
2. Test the temperature of product before using by applying to sensitive skin such as the underside of the wrist.
3. Overheating of inner pack in the microwave can damage or destroy the product. Do not exceed 1 minute or 4 increments of 15 seconds beyond recommended microwave time.
4. If any swelling of inner pack is observed while heating, immediately turn off the microwave and wait for the swelling to go down before opening the oven door.
5. The material contained in the inner pack is non-toxic when used as directed. If the inner pack is accidentally punctured, however, discontinue using the product. If the contents come in contact with the skin, wash with soap and water to remove. Soiled clothing may be laundered or dry-cleaned.
6. Be cautious when using product for people with sensitive skin, such as infants, young children or older persons.
7. Persons with diabetes, circulatory problems, nerve damage, paralysis or insensitive skin should use only as directed by a physician.
8. Consult your doctor before treating any serious injury or if pain persists. Do not use product in conjunction with liniments, balms or lotions. Do not use heating pad after recent surgery unless directed by a physician.

## OPERATING INSTRUCTIONS

**⚠ WARNING:** Misuse can cause severe burns and void warranty. Carefully read the **IMPORTANT SAFETY INSTRUCTIONS** on the inside front page.

Heat therapy relaxes muscles and relieves joint pain and soreness. By increasing blood flow to the injury area, heat therapy can help accelerate healing. Finally, heat can effectively warm and loosen stiff muscles before exercise.

Cold therapy can help decrease swelling due to sprains and strains when applied immediately after injury.

SmartTemp® advanced hot or cold technology maintains optimal therapeutic temperature for over 60 minutes.

SmartTemp provides relief from back, menstrual, arthritis and muscle pain. See Figures 3A-D, 4A-D, and 5A-C. The SmartTemp small and large pad models include an elastic strap to help hold the pad in place for therapy.

SoftHeat Back & Hip wrap\*, as well as the Neck & Shoulder wrap, are specifically designed to contour to your body and hold the heating pad in place. See Figures 4A-D and 5A-C.

\* Fits up to a 47" waist.

### IMPORTANT

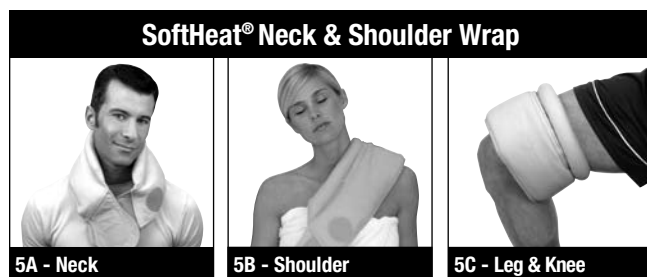
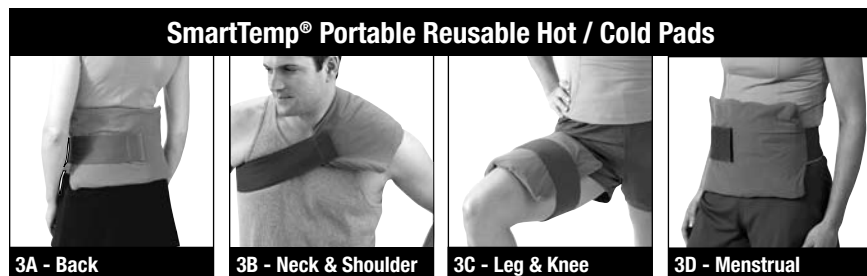
**Please do not return this product to the store.  
Contact Kaz Consumer Relations – we can help you.**

**Call: 1-800-477-0457 weekdays 7:30am – 7:00pm EST**

Email: [consumerrelations@kaz.com](mailto:consumerrelations@kaz.com)


Write: Kaz, Incorporated, Consumer Relations Dept.  
250 Turnpike Rd., Southborough, MA 01772 USA

Canadian residents write to us at:  
Kaz Canada, Inc., Consumer Relations Dept.  
510 Bronte Street South, Milton, ON L9T 2X6 CANADA

**IMPORTANT:** Product must spin on microwave turntable to heat uniformly. If the microwave does not have a turntable, halfway through the heating time rotate product 90 degrees.

**TIP:** Not hot enough? Reheat in microwave for 15 seconds and retest it. Repeat up to 4 times as necessary.

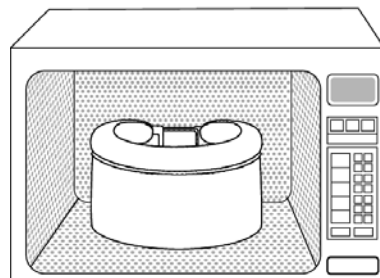**FIG 6** Fold Neck & Shoulder wrap as shown above.

### Dry Heat Therapy

1. Insert inner pack into cloth cover and place in center of microwave. To properly fit in microwave, fold the Large SmartTemp Hot/Cold pad (9" x 14") and the SoftHeat Back & Hip wrap in half. See Fig. 6 for special microwave folding instructions for the SoftHeat Neck & Shoulder wrap.
2. Set the power to HIGH for the time indicated:

PRODUCT	SmartTemp Hot/Cold Pad - Medium (6" x 10")	SmartTemp Hot/Cold Pad - Large (9" x 14")	Back & Hip Wrap	Neck & Shoulder Wrap
TIME 1000W / 1100W Microwave	1 min. 45 sec.	3 min. 15 sec.	2 min. *	3 min. *
TIME 1200W Microwave	1 min. 45 sec. *	3 min. *	1 min.	2 min.

\* Optimal Performance

3. Place product on the area needing heat. See Figures 3A-D, 4A-D, or 5A-C.  
**REHEATING: ⚠ Caution** – When reheating, heat times will be drastically shorter than initial heat up times. Heat in 15 second increments. Repeat up to 4 times until desired temperature is reached.

### Moist Heat Therapy

1. Follow Dry Heat Therapy steps 1 and 2.
2. Dampen one side of the cloth cover with a spray bottle, sponge or cloth.
3. Place damp side of product on the area needing heat. See Figures 3A-D, 4A-D or 5A-C.

### Cold Therapy

1. Place inner pack in the freezer.
2. Chill in freezer for one hour or longer.  
**TIP:** Store in freezer to always have ready.
3. Insert inner pack into cloth cover.
4. Place product on the area needing cold therapy. See Figures 3A-D, 4A-D, or 5A-C.

### Heating From Freezer

Insert SmartTemp hot/cold pad in cloth cover and place in center of microwave. Use one of the following methods to heat from the freezer:

- On DEFROST setting, for
    - SmartTemp Hot/Cold Pad (6" x 10"): 1.3 lbs as the weight; or
    - SmartTemp Hot/Cold Pad (9" x 14"): *(fold in half first)* 2.7 lbs as the weight; or
    - SoftHeat Back & Hip Wrap: *(fold in half first)* 1.5 lbs as the weight; or
    - SoftHeat Neck & Shoulder Wrap: *(fold per FIG 6)* 2.7 lbs as the weight; or
  - On DEFROST setting, use double the amount of time in the chart above, or
  - On POWER setting, use 30% power and double the amount of time in the chart above
- Allow to stand five minutes, then follow normal heating instructions.

## CARE AND CLEANING

1. Remove inner pack.
2. Machine wash cloth cover or wrap in cool water, using mild detergent.
3. Hang dry or tumble dry on a cool setting.
4. Wipe the inner pack clean with a damp cloth, do not use chemicals or place in washer or dryer.

## COMMONLY ASKED QUESTIONS

- Q: My product is not hot enough after following instructions. Can I make it hotter?**
- A:** Yes. If the pad is not hot enough after following the instructions, you may have a microwave rated with lower wattage than used in the instructions. Simply heat product in the microwave for 15 seconds at a HIGH power setting and retest it. Repeat this process up to 4 times until the desired temperature is reached
- Q: Can I store my product in the freezer so I have it ready in an emergency?**
- A:** Yes.
- Q: My SoftHeat back wrap strap will not fasten.**
- A:** The elastic straps are designed to overlap to fasten. Please note the fastening tips of the elastic straps cannot be fastened together. See Figure 2A.
- Q: I have a small waist the SoftHeat back wrap doesn't fit.**
- A:** For smaller waists, simply overlap the elastic straps and attach the top strap to the back of the wrap. See Figure 2A, callout A and D. Please note the fastening tip of the elastic strap cannot attach to the small piece of fabric between the elastic straps and the back of the wrap. See Figure 2A, callout E.

## WARRANTY

This product is warranted against defective material or workmanship from the date of purchase. The length of the warranty is 1 year.

Any defective part will be repaired or replaced at no charge as long as it has not been repaired or tampered with and was used according to these printed instructions. Any charges incurred to ship the product back for replacement are the sole responsibility of the consumer. This warranty gives you specific legal rights as well as other rights, which vary from state to state.

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© 2008