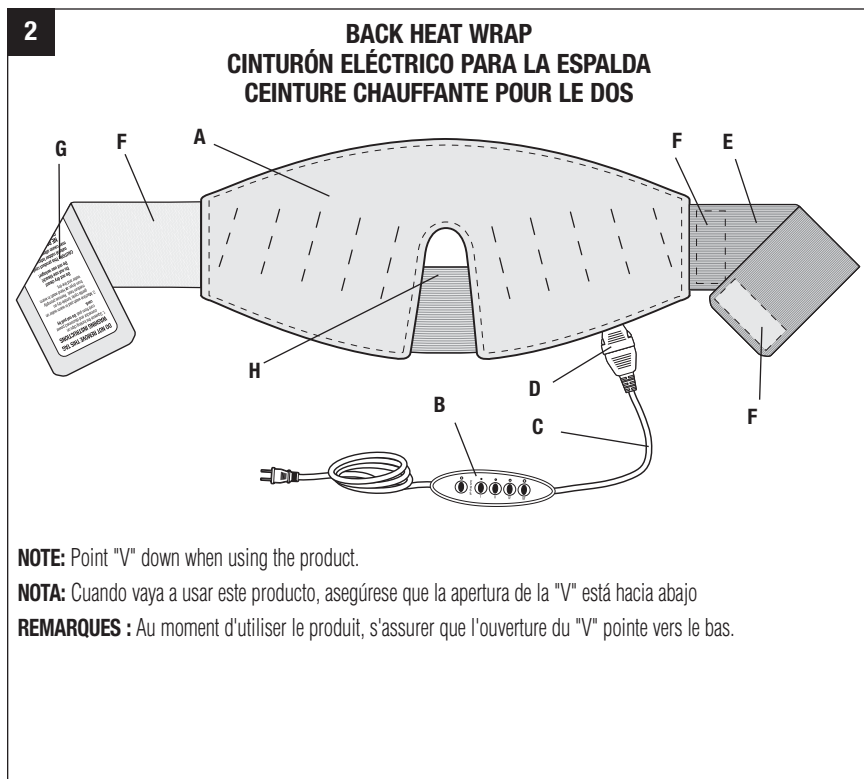
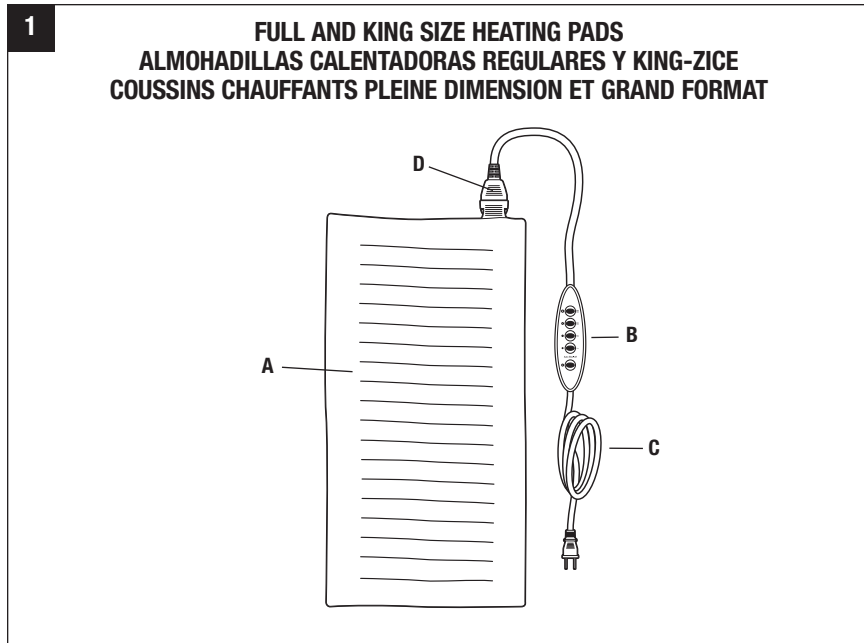
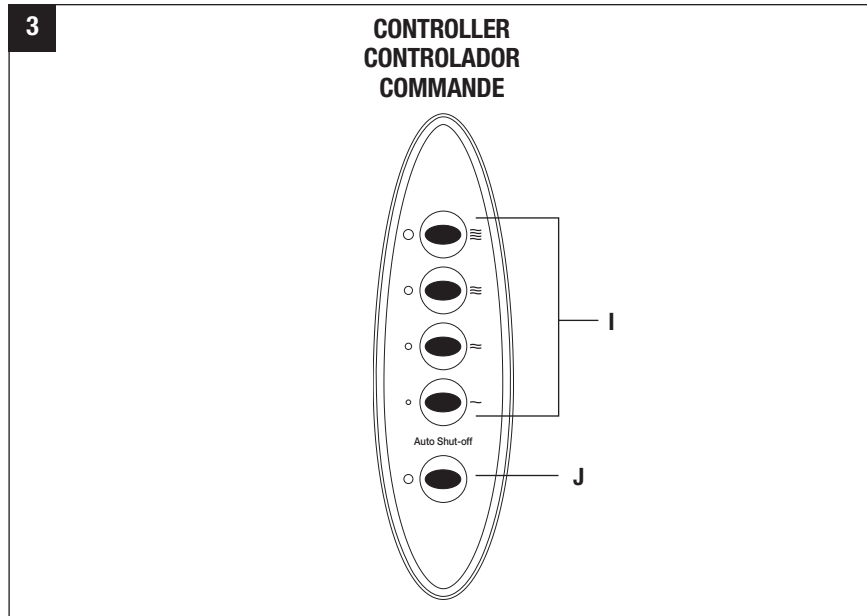
Heating Pad
Almondolla Eléctrica
Coussin Chauffant

CAUTION: The back heat wrap model contains natural rubber latex which may cause allergic reactions.

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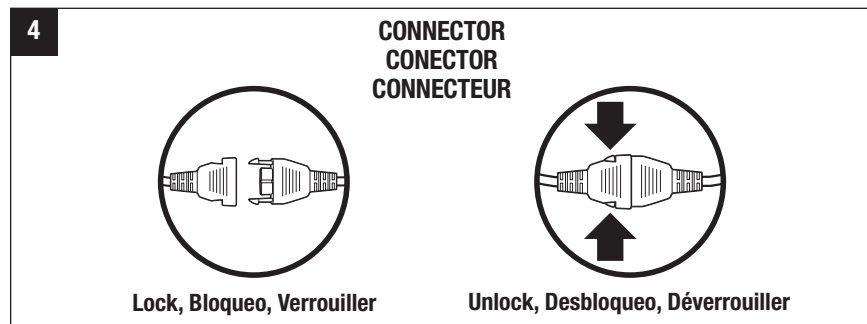
IMPORTANT SAFETY INSTRUCTIONS

DO NOT DESTROY

DANGER: TO REDUCE THE RISK OF SEVERE BURNS, ELECTRIC SHOCK AND FIRE, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS.

1. READ ALL INSTRUCTIONS.
2. DO NOT USE WHILE SLEEPING.
3. DO NOT USE ON AN INFANT.
4. THIS PAD IS NOT TO BE USED ON OR BY AN INVALID, SLEEPING OR UNCONSCIOUS PERSON, OR A PERSON WITH POOR BLOOD CIRCULATION OR DIABETES UNLESS CAREFULLY ATTENDED.
5. DO NOT USE A HEATING PAD WHEN SYMPTOMS OF APPENDICITIS ARE PRESENT. CONSULT WITH YOUR PHYSICIAN.
6. DO NOT USE ON AREAS OF INSENSITIVE SKIN.
7. BURNS CAN OCCUR REGARDLESS OF CONTROL SETTING. CHECK SKIN UNDER PAD FREQUENTLY.
8. DO NOT USE IN AN OXYGEN RICH ENVIRONMENT.
9. **DO NOT USE PINS OR OTHER METALLIC MEANS TO FASTEN THIS PAD IN PLACE.**
10. **DO NOT SIT ON OR CRUSH PAD – AVOID SHARP FOLDS. ALWAYS PLACE PAD ON TOP OF AND NOT UNDER YOUR BODY. NEVER PLACE PAD BETWEEN YOURSELF AND CHAIR, SOFA, BED OR PILLOW.**
11. **NEVER PULL THIS PAD BY THE SUPPLY CORD. DO NOT USE THE CORD AS A HANDLE. UNPLUG WHEN NOT IN USE.**
12. DO NOT USE OUTDOORS.
13. DO NOT USE PAD WITH LINIMENT, SALVE OR OINTMENT PREPARATIONS THAT CONTAIN HEAT-PRODUCING INGREDIENTS. SKIN BURNS COULD RESULT.
14. CAREFULLY EXAMINE PAD BEFORE EACH USE. DISCARD IF PAD SHOWS ANY SIGN OF DETERIORATION.
15. WHEN NOT IN USE, STORE THE PAD IN THIS BOX, IN A COOL, DRY PLACE.
16. DO NOT IMMERSE CONTROLLER IN WATER, PLACE UNDER RUNNING WATER OR PUT IN WASHING MACHINE OR DRYER.
17. SAVE THESE INSTRUCTIONS.

THIS APPLIANCE HAS A POLARIZED PLUG (ONE BLADE IS WIDER THAN THE OTHER). AS A SAFETY FEATURE, THIS PLUG FITS A POLARIZED OUTLET ONLY ONE WAY. IF THE PLUG DOES NOT FIT FULLY IN THE OUTLET, REVERSE THE PLUG. IF IT STILL DOES NOT FIT, CONTACT A QUALIFIED ELECTRICIAN. DO NOT ATTEMPT TO DEFEAT THIS SAFETY FEATURE.

OPERATING INSTRUCTIONS

Heat therapy relaxes muscles and relieves joint pain and soreness. By increasing blood flow to the injury area, heat therapy can help accelerate healing. Finally, heat can effectively warm and loosen stiff muscles before exercise.

Ultra's advanced moist or dry technology delivers the best heat control and timed-release moisture technology for physician recommended moist heat. Simply dampen the pad for moist heat. Moist heat provides deeper heat penetration than dry heat alone.

Kaz Ultra products are also UltraSoft and easy to clean – just disconnect the controller and machine wash and dry the entire pad.

Ultra therapy products provide relief from back, menstrual, arthritis and muscle pain. Please note that the pad must be placed on the injured area, but not under your body, for safe operation. See Figures 5A-D.

The Kaz adjustable back heat wrap is designed to contour to your back for therapy. One size fits all*. The heat wrap can also be used for menstrual pain, shoulder and neck pain and hip pain. See Figures 6A-D.

* Fits up to a 47" waist.

IMPORTANT

**Please do not return this product to the store.
Contact Kaz Consumer Relations – we can help you.**

Call: 1-800-477-0457 weekdays 7:30am – 7:00pm EST

Email: consumerrelations@kaz.com

Write:

**Kaz, Incorporated
Consumer Relations Dept.
250 Turnpike Rd.
Southborough, MA 01772 USA**

**Kaz Canada, Incorporated
510 Bronte Street South
Milton, ON L9T 2X6 CANADA**

FULL AND KING SIZE HEATING PADS

5A - Back

5B - Menstrual

5C - Shoulder

5D - Leg

BACK HEAT WRAP

6A - Back

6B - Menstrual

6C - Shoulder and Neck

6D - Hip

Dry Heat Therapy

1. Plug the power cord into a 120 volt AC electrical outlet.
2. Set the controller to desired setting. See Figure 3.
3. Place heating pad on the area needing heat. See Figures 5A-D or 6A-D.
4. Press the OFF Button and unplug power cord when finished.

Moist Heat Therapy

1. Unplug pad from wall outlet.
2. Dampen one side of the heating pad with a spray bottle, sponge or cloth.
 - a. **DO NOT immerse heating pad in water or place under running water.**
 - b. **DO NOT spray water on the connector (See Figure 1 and 2).**
3. Follow Dry Heat Therapy steps 1 and 2 above.
4. Place dampened side of heating pad on the area needing heat. See Figures 5A-D or 6A-D.
5. Press the OFF Button and unplug power cord when finished.

Automatic Shut-off

Your heating pad is equipped with a special safety feature designed to shut the heating pad off after 60 minutes of use. When the automatic shut-off feature turns the pad off, the red indicator light on the controller flashes slowly to let you know the pad has shut off. To restart the pad, just reselect the desired heat setting. See Figure 3.

CARE AND CLEANING

DO NOT DRY CLEAN! DO NOT USE BLEACH! DO NOT USE WRINGER!

1. Set controller to OFF. Unplug pad from wall outlet.
2. Squeeze the locking clips on connector and disconnect power cord. See Figure 4.
DO NOT pull by the cord.
3. Machine wash pad in warm water on gentle cycle; tumble dry on medium heat. Remove promptly from the dryer. Or, hand wash in warm water and line dry.
4. **DO NOT** plug in pad until it is dry.

COMMONLY ASKED QUESTIONS

Q: I was using my heating pad and now it isn't producing heat and the controller is flashing.

A: This heating pad is equipped with an automatic shut-off safety feature to shut itself off after 60 minutes. To restart the pad, just reselect your desired heat setting. See Figure 3.

Q: How fast should my heating pad heat-up?

A: Your heating pad is specially designed for the fastest heat-up allowed by Underwriters Laboratories (UL), the independent testing lab that the electronics industry relies on for safety testing. Your heating pad should reach the high temperature setting in less than 10 minutes.

Q: My heating pad doesn't feel hot enough?

A: Your heating pad is specially designed for the highest heat allowed by Underwriters Laboratories (UL), the independent testing lab that the electronics industry relies on for safety testing.

Q: How come I can't lay or sit on my heating pad?

A: Laying or sitting on the pad could damage the wires in the pad. It's also very important to avoid folding or creasing the pad. Additionally, the pad needs to have one side exposed to the open air at all times so it can breathe for safe operation. Using the pad any other way could cause damage to the pad or cause severe burns.

Q: I have a small waist the back heat wrap doesn't fit.

A: With the washing label facing your body, wrap the elastic strap around your waist and attach the fastening location on the tip of elastic strap to the heating pad. See Figure 2, callouts A, F and G.

WARRANTY

This heating pad is warranted against defective material or workmanship from the date of purchase. The length of the warranty is 5 years.

Any defective part will be repaired or replaced at no charge as long as it has not been repaired or tampered with and was used according to these printed instructions. Any charges incurred to ship the product back for replacement are the sole responsibility of the consumer. This warranty gives you specific legal rights as well as other rights, which vary from state to state.

CONTACT US

If you experience unsatisfactory operation or have any questions or comments, call the Kaz Consumer Relations Department at 800-477-0457 or write to:

Kaz, Incorporated
Consumer Relations Dept.
250 Turnpike Rd.
Southborough, MA 01772 USA

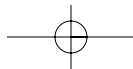
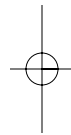
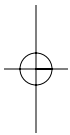
Kaz Canada, Inc.
510 Bronte Street South
Milton, ON L9T 2X6 Canada

You can also email Kaz at consumerrelations@kaz.com.

Please be sure to include the model number of the unit, as well as a detailed description of how the pad is operating.

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© 2006 Kaz, Incorporated